

Gebrauchsanweisung für das Pflaster

Activ'M© is a silicone self-adherent dressing made of a polyurethane film and a fine layer of silicone gel. These components are selected for their biocompatibility.

USES

Intrinsic advantages of silicone gel improve the appearance of old and new scars.

- To cure hypertrophic or keloid old or new scars
- To prevent close wounds from becoming keloid or hypertrophic scars.

CONTRAINDICATIONS

Activ'M© cannot be used in the following cases:

- open wounds,
- maceration
- burns,
- mucose,

In the event of a skin reaction, stop treatment and consult your doctor.

SECONDARY EFFECTS

Side effects are unknown to date.

DIRECTIONS

- Before use, cleanse and thoroughly dry the scar.
 - Remove the dressing from its individual pack. The dressing can be cut. Make sure the dressing completely covers the scar as well as the surrounding area (at least 1/2 cm on all sides of the scar).
 - Start by removing the protective film from the dressing and place the adhesive side at the edge of the scar.
- Ideally cleanse and dry the scar once a day with a neutral soap.
- The dressing should be worn for 24 hours a day.
 - Change the dressing every 3 to 5 days when it no longer sticks. Do not wear the same dressing for over 7 days.